

DNTL | Dental Division

A Safe to handle, Unique & Patented Biotechnology based, Non-Toxic, Non-Harsh, Non-Corrosive, Non-Carcinogenic, Environment Friendly, Biodegradable, certified Food Contact No Rinse **Chemicals**



3485 Ashby, Ville Saint Laurent
Quebec, H7W 2N9, Canada.
Tel: +1 (514) 745 2597
Fax: +1 (514) 745 5176
www.atomesbio.com

SCOPE OF APPLICATION

HANDS CLEANING

HANDS PRE-CHIRURGICAL DISINFECTION

EVACUATION SYSTEM CLEANER

MEDICAL TOOLS & EQUIPMENTS | Cleaning & Disinfection

OPERATION ROOMS | Cleaning & Disinfection

OPEN SURFACES | Cleaning & Disinfection

HANDS CLEANING

Perly | Antiseptic hand soap at pH neutral

- ✓ Highly foaming antiseptic hands cleaner used for routine hand washing and medical hand washing; sanitizing and disinfection.
- ✓ Ready to use, can be conveniently used from the dispenser, 4ml to 7ml should be adequate for cleaning hands. Rinse with abundant water. **Perly** can be applied from any kind of soap dispenser.
- ✓ Combination of various surfactants, Benzalkonium chloride (BAC).
- ✓ pH 7.00
- ✓ Odor pleasant.
- ✓ Leaves hands soft and smooth.
- ✓ Rapid & powerful wetting characteristics.
- ✓ Instant & complete solubility.



HANDS PRE-CHIRURGICAL DISINFECTION

Anti-microbe Din# 02248351 | Alcohol free instant hand sanitizer

- ✓ **Alcohol Free hand sanitizer** No Cracks and hands are smooth and soft and non-flammable.
- ✓ **Certified Food Contact No Rinse** Safer, Benzalkonium chloride (BAC).
- ✓ **Instant hand foaming sanitizer** in 15 seconds it kills 99.99% of most disease causing germs specially the ones related to SARS, colds flu, H1N1 and all C difficile bacteria.
- ✓ **Extended Retention time** Protects more than 60 min post application
- ✓ **DIN Number (Drug Identification Number)** Classified as a drug (Unique in its category)
- ✓ **Extremely cost-efficient** 0.4 ml / application | 1 Liter equivalent to 2000 applications



HANDS PRE-CHIRURGICAL DISINFECTION

Neutra-chlorexidine | Alcohol-free skin antiseptic, based on stable formulation of chlorhexidine gluconate 2% w/v

- ✓ Non-carcinogenic chlorhexidine gluconate 2% based skin antiseptic.
- ✓ Environment-friendly.
- ✓ Alcohol-free
- ✓ pH neutral, non-flammable, non-toxic.
- ✓ Stable, non-irritant and effective against a wide range of Gram-positive and Gram-negative vegetative bacteria including MRSA1 and MSSA1, yeasts, dermatophyte fungi and lipophilic viruses.
- ✓ Usage: Pre-operative surgical hand disinfection, Antiseptic hand wash on the ward, Pre-operative skin antiseptic for the patient, Post-operative skin antiseptic for the patients and Children and elderly patients.



EVACUATION SYSTEM | Cleaning & Disinfection

Bio-Flux | 2 in 1, cleaner and disinfectant for evacuation system.

- BIO FLUX dissolves the inorganic deposits and other residues (i.e. Scale) of your evacuation system and keeps it clean and deodorized.
- Environment-friendly.
- Dilute Bio Flux from 1.5% to 3% (15 ml/L to 30 ml/L)
- Use 15 ml to 30 ml BIO FLUX per treatment.
- Aspirate the prepared solution by the saliva ejector and the high suction system.
- Use this solution also for the spittoon.



MEDICAL TOOLS & EQUIPMENTS | Cleaning

Evolutionase P | Neutral foaming enzymatic detergent

- ✓ Blend of Amylase, Protease and Lipase and surfactants having wetting, penetrating and cleaning properties.
- ✓ Very effective in removing grease on hard surfaces. The presence of stable-protease-Lipase-Amylase enzymes in the formulation makes **Evolutionase P** highly efficient in removing blood and protein deposits.
- ✓ Neutral product
- ✓ Removes grease efficiently
- ✓ Removes blood and protein deposits efficiently
- ✓ Eliminates harsh and dangerous cleaning chemicals and solvents.



MEDICAL TOOLS & EQUIPMENTS | Cleaning

Aci Quat DIN# 02244321 | 2-in-1 cleaner & disinfectant for medical equipments

- ✓ Belongs to a family of polyquats that are environmentally friendly.
- ✓ Wide spectrum: Virucidal, bactericidal, fungicidal, algaecidal and tuberculocidal.
- ✓ 6 active ingredients against micro-organisms.
- ✓ Neutral product
- ✓ Removes grease efficiently
- ✓ Removes blood, protein and inorganic deposits efficiently from medical equipments and endoscopes.
- ✓ Eliminates harsh and dangerous cleaning chemicals and solvents.
- ✓ DIN Number (Drug Identification Number) Classified as a drug (Unique in its category)



MEDICAL TOOLS & EQUIPMENTS | Disinfection & Sterilization

Bioxy S | pH neutral disinfection and cold sterilization for endoscopes, medical and surgical instruments

- ✓ Solid peracetic acid based chemical sterilant, Once diluted in water at 1%, yields 2000 ppm of active peracetic acid for a sporocidal action in 10 minutes at pH 7.00
- ✓ Wide spectrum: Virucidal, bactericidal, fungicidal, algacidal, tuberculocidal and sporocidal including MRSA, Pseudomonas aeruginosa, Escherichia coli, mycobacterium, fungi, virus (HIV,HCV,HBV), spores (B. subtilis endospores, Clostridium difficile)
- ✓ Eliminates harsh and dangerous cleaning chemicals and solvents.
- ✓ Non-corrosive, non-harsh, **Bioxy S** can be applied on all types of endoscopes, on metals (stainless steel, aluminum), ceramics, polymers (polyethylene, polyester, polypropylene, Plexiglas, Teflon), adhesives and rubbers.



MEDICAL SURFACES | Disinfection & Sterilization

Multi Action 5 DIN# 02264021 | Ready to use, pH neutral wide spectrum disinfectant for surfaces in low and high-risk zones


- ✓ Belongs to a family of polyquats that are environmentally friendly.
- ✓ Wide spectrum: Virucidal, bactericidal, fungicidal, algacidal, tuberculocidal and sporocidal
- ✓ 5 active ingredients against micro-organisms.
- ✓ Neutral product
- ✓ Eliminates harsh and dangerous cleaning chemicals and solvents.
- ✓ DIN Number (Drug Identification Number) Classified as a drug (Unique in its category)
- ✓ Non-corrosive, non-harsh, **Multi Action 5** can be applied on all types of endoscopes, on metals (stainless steel, aluminum), ceramics, polymers (polyethylene, polyester, polypropylene, Plexiglas, Teflon), adhesives and rubbers.



OPERATION ROOMS | General Disinfection & Fumigation

Ato Quat DIN# 02253658 | Neutral disinfectant and air fumigation in operation room and high-risk zone

- ✓ Environment-friendly, pH neutral, non-flammable, non-toxic, non-carcinogenic disinfectant, based on quaternary ammonium compounds, for air fumigation in operation room and high-risk zones (Intensive care unit, operation room, isolation room, Coma patient room).
- ✓ Non-corrosive on equipments and non-irritant to skin.
- ✓ A broad germicidal activity against a variety of: Gram-positive and gram-negative, Yeast, mold and algae, Virus such as Hepatitis B Virus, Hepatitis C Virus, Herpes Simplex virus Type 1, Herpes Simplex virus Type 2, HIV-1 (AIDS Virus), Influenza A / Brazil (Influenza), Respiratory Syncytial Virus (RSV), Vaccinia (Pox Virus).
- ✓ DIN Number (Drug Identification Number) Classified as a drug (Unique in its category)

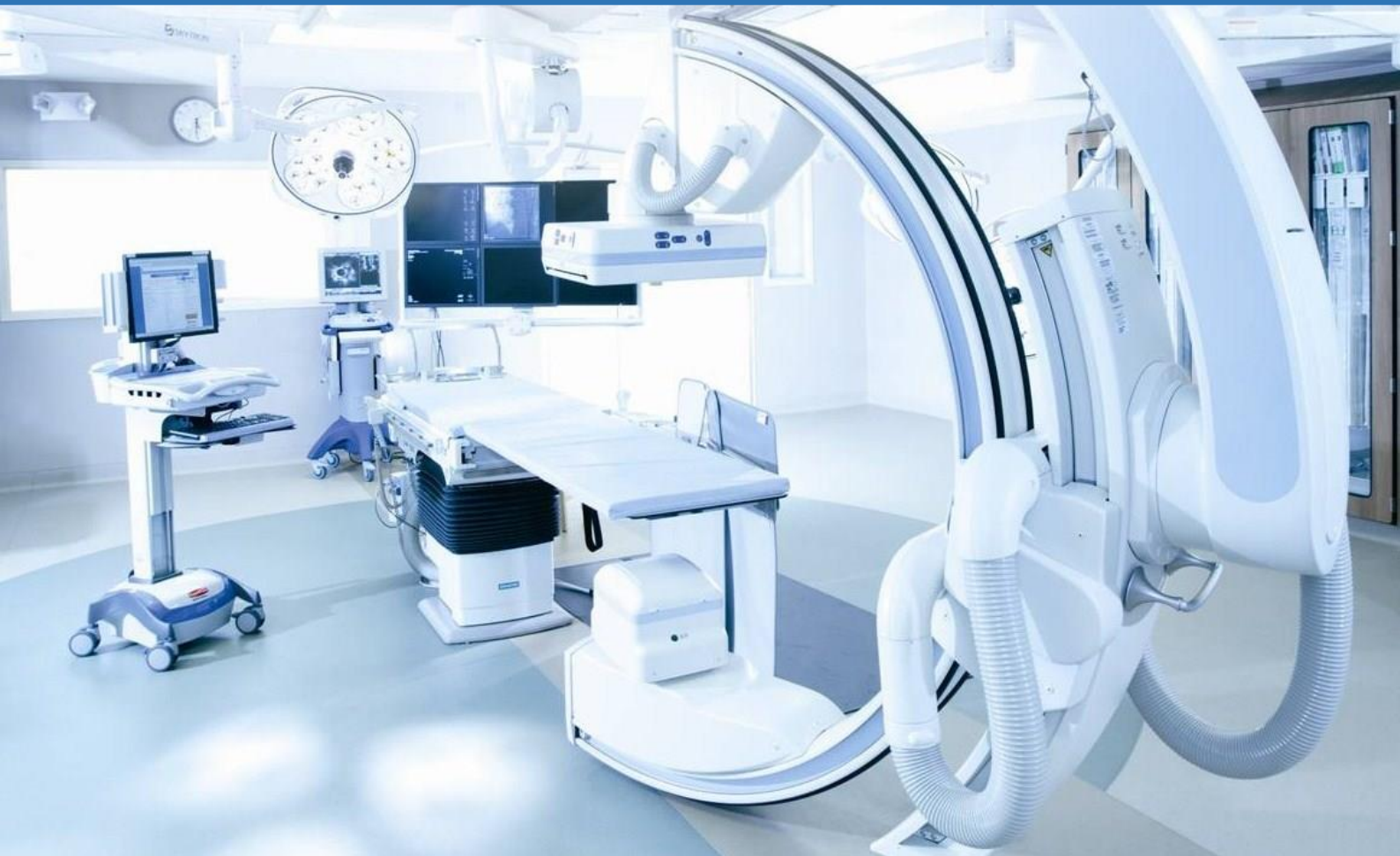

OPEN SURFACES | Cleaning & Disinfection

Sol Klin | 2-in-1 Neutral cleaner and disinfectant for floors, offices and general areas

- ✓ Environment-friendly, pH neutral, non-flammable, non-toxic, non-carcinogenic disinfectant, based on quaternary ammonium compounds.
- ✓ 2-in-1 cleaning and disinfection.
- ✓ A wide spectrum disinfectant
- ✓ Virucidal, Bactericidal, Fungicidal and Algaecidal.
- ✓ Safe on wood.
- ✓ Extremely cost-efficient 3% as disinfectant.
- ✓ Removes inorganics (i.e. scale) and organics (i.e. food) while killing microorganisms.
- ✓ Non-corrosive on equipments and non-irritant to skin.

